



UPSTATE
MEDICAL UNIVERSITY

Materials Management

Clinical Supplies, Materials and Logistics



**30g x 1.5mm Pressure Activated Safety Lancet
Supplier Change (Lawson #76141):**

Old Product:

BD Purple 30g x 1.5mm Lancet



New Product:

PremierPro Green 30g x 1.5mm Lancet



This product functions in the same way as the old BD product. See instructions for use below:

1. Twist off the protective cap and pull it straight out.
2. Press the safety lancet body firmly against the puncture site to activate.
3. Gently apply intermittent pressure near the puncture site to obtain required blood volume.