



UPSTATE MEDICAL UNIVERSITY

Materials Management

Clinical Supplies, Materials and Logistics



Baxter Secondary Medication Tubing Set (Lawson #23144):

The following changes have been made by the manufacturer:

- Spike tip protector cap changed from vented to non-vented design. NOTE: Priming will not complete with the spike cap on since the design is now non-vented. Remove the cap to back-prime the tubing from the primary fluid line.
- Distal male Luer changed to two-piece Luer with retractable collar. When using, retract the collar and grasp the rigid stem (not the collar). Insert the stem with a firm push and twist motion, and then engage the collar.

